Thompson Retractor


Infinite Height Rail Clamp

Low Profile, Uncompromised Exposure

The Infinite Height Rail Clamp (IHRC) can be easily adjusted to any height between 13" - 22", allowing versatility for multiple procedures and patient sizes. The adjustable design prevents the post from interfering with operating space. The stable design can be used for both unilateral and bilateral set ups.


Choose an Infinite Height Rail Clamp with one or two cam joints to replace any standard Elite II Rail Clamp for unilateral or bilateral configurations.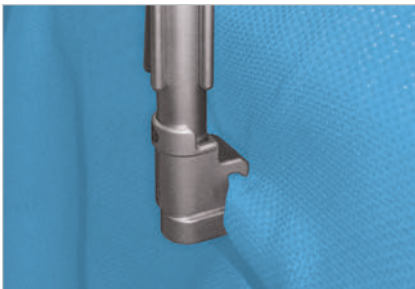


Step 1

Open rail clamp jaw and place on bedrail over the sterile drapes.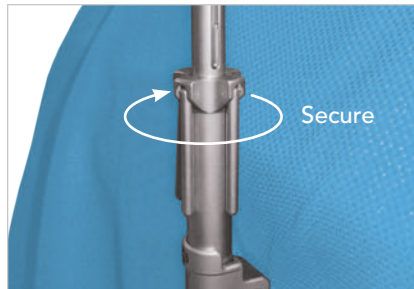

Step 2

Rotate the knob at the bottom of the rail clamp clockwise to tighten, utilizing the attached hanging handles for leverage when needed.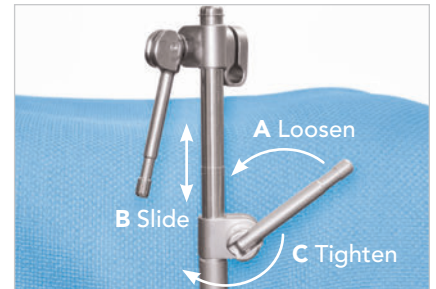


Step 3

Adjust height by turning the cam joint counter-clockwise to loosen (A). Slide the post to the desired height (B) and turn the joint clockwise to tighten (C).

NOTE:

Do not attempt to adjust rail clamp height with frame components attached unless supporting the weight of the frame components with a hand. The post will not slide easily during height adjustments if this is not done.

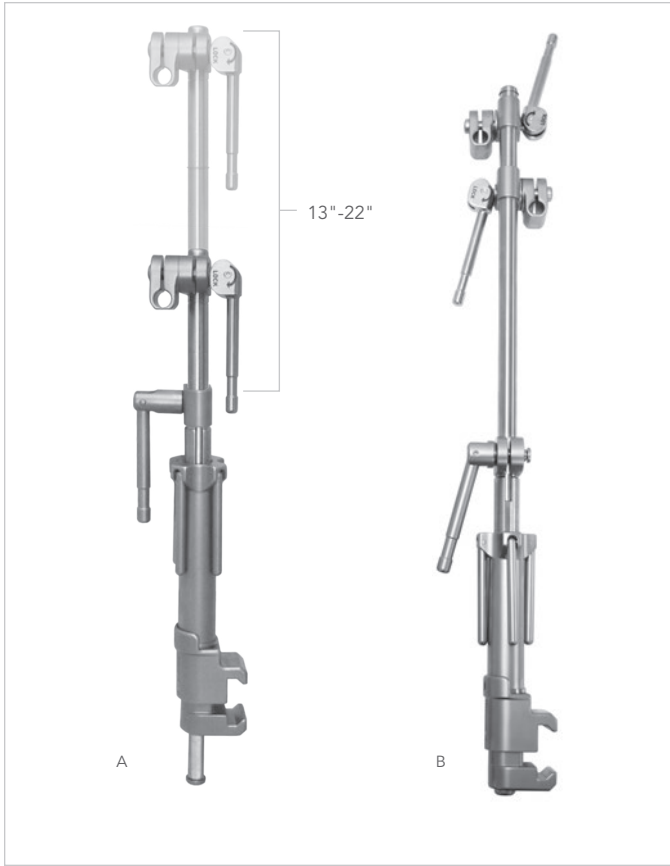INFINITE HEIGHT RAIL CLAMPS (IHRC)

REF	ITEM DESCRIPTION	PART #
A	IHRC with 1 Cam Joint	41902IRC
B	IHRC with 2 Cam Joints	41903IRC

IHRC FOR COMPETITOR SYSTEMS

REF	ITEM DESCRIPTION	PART #
C	IHRC with 1 Cam Joint for Omni-Tract	41908IRC
D	IHRC with 1 Cam Joint for Bookwalter	41909IRC

Thompson
 SURGICAL INSTRUMENTS
 10341 East Cherry Bend Road
 Traverse City, Michigan 49684
 phone: 231.922.0177
 fax: 231.922.0174
thompsonsurgical.com

EC REP **Emergo Europe**
 Prinsessegracht 20
 2514 AP The Hague
 THE NETHERLANDS

* Free trial valid for U.S. customers only. International customers please call for more information.

© 2017 Thompson Surgical Instruments, Inc. Traverse City, Michigan.

© S-Lock®, PLA®, and the "T Circle" logomark are Registered Trademarks of Thompson Surgical Instruments, Inc.
 Patents: US4971038, US5025780, US5888197, US5897087, US5902233, US5984865, US6033363, US6416465, US6511423, US7338442, US7749163, US8257255, US8360971, US8617064
 Other patents pending.

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