



# THOMPSON RETRACTOR USER MANUAL

## MIS Retractor Lite LED

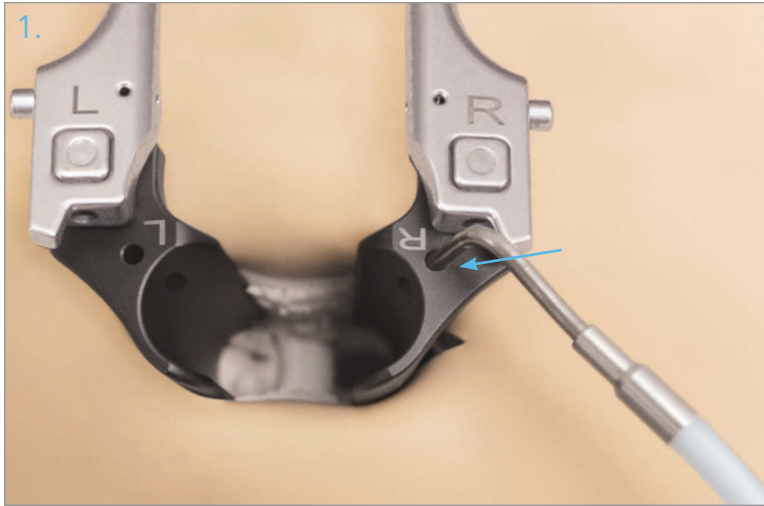

1. Insert the distal ends of the light into the blade channels.

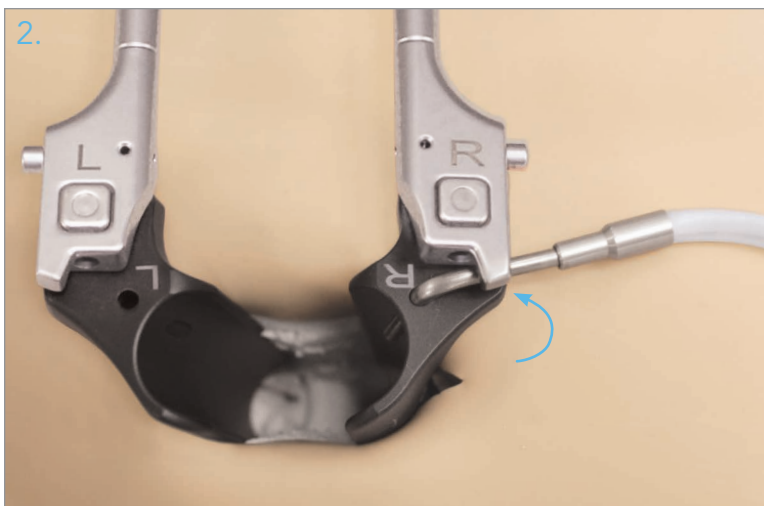

2. Swing the light ends back so that they hook under the frame arms securely.

TIP:

Do not pull on cable during this step, apply pressure to rigid, metal portion of distal end to avoid damaging cable.

TIP:

More or less pressure may be needed to secure the light ends under the frame arms, variations in pressure needed are normal.

3. Plug light cable in to light source and turn light source on.

**REMOVAL:** DO NOT remove lights from blades without first turning the light source off. Please refer to MIS Retractor Lite LED IFU for all precautions. Gently apply pressure to unhook the light ends from the frame arms and slide light ends out of blade.

**Thompson**   
SURGICAL INSTRUMENTS

10170 East Cherry Bend Road  
Traverse City, Michigan 49684  
phone: 231.922.0177  
fax: 231.922.0174

[thompsonsurgical.com](http://thompsonsurgical.com)