

Mini
Retractor
Lite LED
User Manual

Thompson Retractor 



Uncompromised Exposure.

Please read the Mini Retractor Lite LED IFU for more information.


STEP 1

Press distal end of Mini Retractor Lite (40002MRL) into Mini Lite Clip (40026MRL). Apply pressure to the rigid metal portion of the light only to avoid damaging cable.



STEP 2

Slide Mini Lite Clip and Mini Retractor Lite on to retractor blade (light is intended to be used with Thompson Anterior Cervical Blades, see Mini Retractor Lite IFU for exceptions). Adjust clip so that the light emitting end of Mini Retractor Lite is unobstructed and points down blade face.

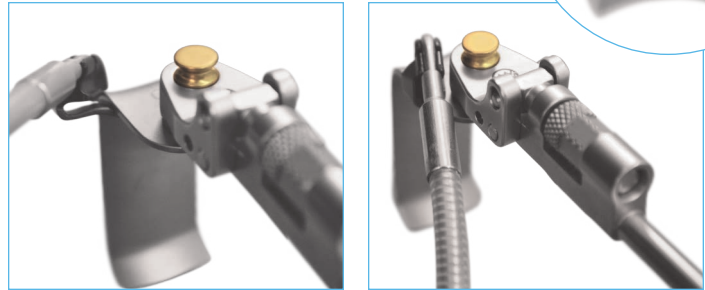


STEP 3

Plug Mini Retractor Lite cable into LED Light Source (LLS-2000) and turn source on.

STEP 4

Avoid moving Mini Retractor Lite cable and reduce cable tension to prevent clip from moving on the blade.

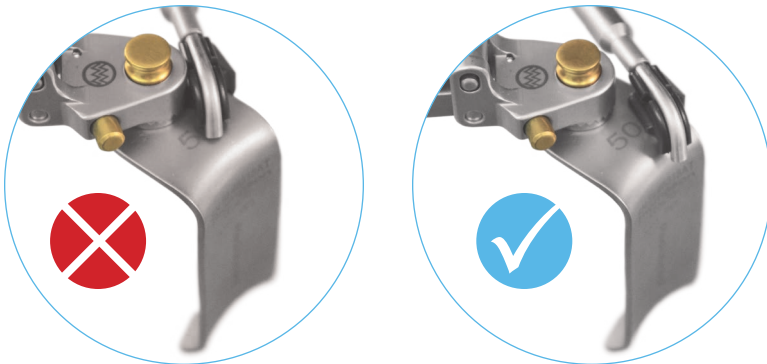

OPTION

S-Lock may be utilized to rotate blade without affecting clip position in most cases. If clip position is affected, move clip and light to a different blade.



Thompson 
SURGICAL INSTRUMENTS

10170 East Cherry Bend Road
Traverse City, Michigan 49684
phone: 231.922.0177
fax: 231.922.0174

thompsonsurgical.com

**For a FREE TRIAL
call 1.800.227.7543**

Free trial valid for U.S. customers only.
International customers please call for more information.

Made in the USA