

Thompson Retractor

S-Lock® Nathanson Hooks

Eliminate Slippage and Rotation

The S-Lock Nathanson Hook is used for organ retraction. Thompson offers the hook in three sizes: small, medium, and large, each available in either 5mm or 6.5mm diameters.

- Quickly connects and disconnects
- S-Lock eliminates slippage and rotation
- Frees up staff



"Hands down, this should be every surgeon's go-to retractor."

- Michael Nizzi, DO, FACOS




Step 1

If using S-Lock Rigid Laparoscopic Holder, skip ahead to Step 2.

After assembling desired laparoscopic holding arm, insert S-Lock Nathanson Adapter into arm's distal end. (A)

Secure into place by turning knurled knob on laparoscopic holding arm clockwise until tight. (B)

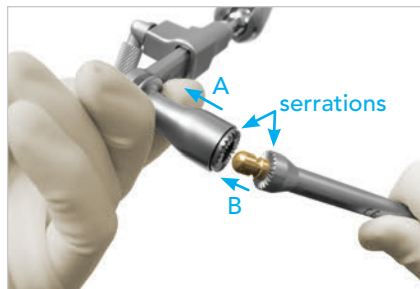

Step 2

Pull back S-Lock plunger fully (A) and insert S-Lock Nathanson Hook nipple, aligning serrations (B).




Step 3

Once serrations are aligned, release plunger to hold in place, hands-free and retract as needed.



NOTICE:

Non-S-Lock Nathanson Hooks available, call for information.

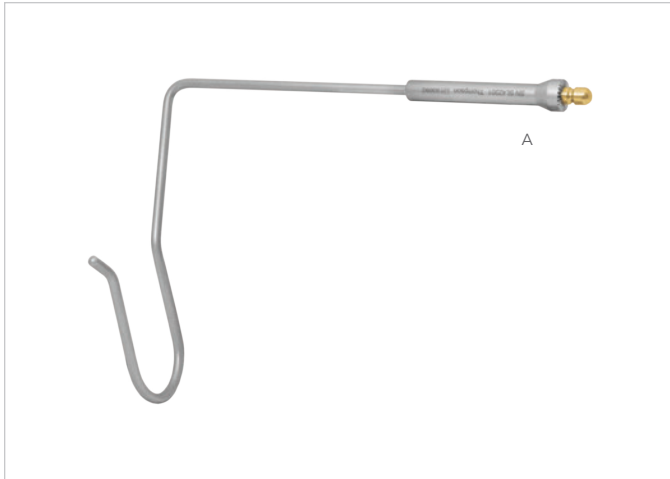

S-LOCK® NATHANSON HOOKS

REF	ITEM DESCRIPTION	PART #
	Small 5mm (169mm x 125mm x 12mm)	SL42200
A	Medium 5mm (203mm x 167mm x 16mm)	SL42201
	Large 5mm (240mm x 222mm x 21mm)	SL42202
	Small 6 ½mm (169mm x 125mm x 12mm)	SL42203
	Medium 6 ½mm (203mm x 167mm x 16mm)	SL42204
	Large 6 ½mm (240mm x 222mm x 21mm)	SL42205



S-LOCK RIGID LAPAROSCOPIC HOLDERS

REF	ITEM DESCRIPTION	PART #
B	S-Lock Rigid Laparoscopic Holder for Elite (or IHRC)	SL42139E
C	S-Lock Rigid Laparoscopic Holder for PRC	SL42139N



S-LOCK® NATHANSON ADAPTER

REF	ITEM DESCRIPTION	PART #
D	S-Lock Nathanson Adapter (required for use with Articulating Arm or Flexible Laparoscopic Holder)	SL42161

NOTICE: The S-Lock Nathanson Adapter attaches to Thompson Articulating Arm and Flexible Laparoscopic Holder to allow for S-Lock Nathanson attachment.

Thompson
SURGICAL INSTRUMENTS
10341 East Cherry Bend Road
Traverse City, Michigan 49684
phone: 231.922.0177
fax: 231.922.0174
thompsonsurgical.com

EC REP **Emergo Europe**
Prinsessegracht 20
2514 AP The Hague
THE NETHERLANDS

* Free trial valid for U.S. customers only. International customers please call for more information.

© 2017 Thompson Surgical Instruments, Inc. Traverse City, Michigan.

© S-Lock®, PLA®, and the "T Circle" logomark are Registered Trademarks of Thompson Surgical Instruments, Inc.
Patents: US4971038, US5025780, US5888197, US5897087, US5902233, US5984865, US6033363, US6416465, US6511423, US7338442, US7749163, US8257255, US8360971, US8617064
Other patents pending.

Rev B
071317
slnh

CE0297
Made In USA