

Thompson Retractor

S-Lock® Nathanson Hooks

Eliminate Slippage and Rotation

The S-Lock Nathanson Hook is used for organ retraction. Thompson offers the hook in three sizes: small, medium, and large, each available in either 5mm or 6.5mm diameters.

- Quickly connects and disconnects
- S-Lock eliminates slippage and rotation
- Frees up staff



"Hands down, this should be every surgeon's go-to retractor."


- Michael Nizzi, DO, FACOS



Step 1

After assembling desired laparoscopic holding arm, insert S-Lock Nathanson Adapter into arm's distal end. (A)

Secure into place by turning knurled knob on laparoscopic holding arm clockwise until tight. (B)

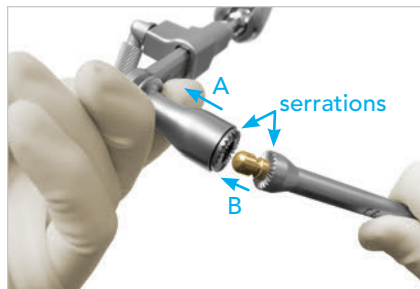

Step 2

Pull back S-Lock Nathanson Adapter plunger fully (A) and insert S-Lock Nathanson Hook nipple (B).



Step 3

Once serrations are aligned, release plunger to hold in place, hands-free and retract as needed.



NOTICE:

The S-Lock Nathanson Adapter (#SL42161) attaches to Thompson Articulating Arm (#42138LE) and Flexible Laparoscopic Holder (#42133DEN) to allow for S-Lock Nathanson attachment.



S-LOCK® NATHANSON ADAPTER

REF	ITEM DESCRIPTION	PART #
A	S-Lock Nathanson Adapter (required for use)	SL42161

S-LOCK® NATHANSON HOOKS

REF	ITEM DESCRIPTION	PART #
	Small 5mm (169mm x 125mm x 12mm)	SL42200
B	Medium 5mm (203mm x 167mm x 16mm)	SL42201
	Large 5mm (240mm x 222mm x 21mm)	SL42202
	Small 6 ½mm (169mm x 125mm x 12mm)	SL42203
	Medium 6 ½mm (203mm x 167mm x 16mm)	SL42204
	Large 6 ½mm (240mm x 222mm x 21mm)	SL42205

 **NOTICE:** Original non-S-Lock Nathanson Hooks are still available. Call for information.


 **Thompson**
 SURGICAL INSTRUMENTS
 10170 East Cherry Bend Road
 Traverse City, Michigan 49684
 phone: 231.922.0177
 fax: 231.922.0174
thompsonsurgical.com

EC REP Emergo Europe
 Prinsessegracht 20
 2514 AP The Hague
 THE NETHERLANDS

Rev A
 021617
 slnh0217

* Free trial valid for U.S. customers only. International customers please call for more information.

© 2016 Thompson Surgical Instruments, Inc. Traverse City, Michigan. Printed in U.S.A.

© S-Lock®, PLA®, and the "T Circle" logomark  are Registered Trademarks of Thompson Surgical Instruments, Inc.
 Patents: US4971038, US5025780, US5888197, US5897087, US5902233, US5984865, US6033363, US6416465, US6511423, US7338442, US7749163, US8257255, US8360971, US8617064
 Other patents pending.

CE0297

Made In USA