Thompson

SURGICAL INSTRUMENTS

Instructions For Use Lite Wand LED and Retractor Lite LED

CE 0297

Distributed by:

Thompson Surgical Instruments, Inc.
10170 E. Cherry Bend Rd.
Traverse City, MI 49684

Customer Service: 231-922-0177
www.thompsonsurgical.com

Sunoptic Technologies®
6018 Bowdendale Avenue
Jacksonville, FL 32216 USA

RMS UK, Ltd.
28 Trinity Road
Nailsea, Somerset BS48 4NU
United Kingdom
TEL: 01275 858891

PLEASE READ BEFORE USE

Failure to follow these instructions could render the device unusable and void any warranty.
--

DEVICE DESCRIPTION/DEVICE CLASSIFICATION

Light Wand LED and Retractor Lite LED – Class II US FDA, Class IIa EU

Thompson Surgical Instruments, Inc. fiberoptic light guides are designed to deliver maximum light when coupled to a medical grade fiberoptic light source.

The Lite Wand LED and Retractor Lite LED may be used with a LED lightsource only and are only compatible with a specific LED lightsource manufactured by Sunoptic Technologies® and distributed by Thompson Surgical Instruments. The Lite Wand LED and Retractor Lite LED cable includes a rigid portion which is compatible with the Thompson Retractor and a flexible portion serves to orient the position of the Lite Wand LED and Retractor Lite LED.

INTENDED USE

The Lite Wand LED and Retractor Lite LED are intended to illuminate a surgical site by relaying light from a fiberoptic light source on to a desired site. Thompson Surgical Instruments, Inc. light guides, such as the Lite Wand LED and Retractor Lite LED, are medical grade, high transmittance, peak efficiency.

CONTRAINDICATIONS

Do not use the Lite Wand LED and Retractor Lite LED with xenon, halogen, or non compatible LED lightsources.

WARNINGS & PRECAUTIONS

Federal law restricts this device to sale by or on the order of a licensed practitioner.

Not suitable for use in presence of flammable anesthetic mixture with air or with oxygen or nitrous oxide.

The user of this product should be thoroughly trained and familiar in the use and care of the product. Use caution and treat the Lite Wand LED and Retractor Lite LED as you would any fine optical device.

Lite Wand LED and Retractor Lite LED are provided non-sterile and must be sterilized before use. See instructions for cleaning and sterilization.

Always inspect the Lite Wand LED and Retractor Lite LED and do not use if there is any evidence of damage. Pay particular attention to the optical surfaces looking for scratches or dings. Pay particular attention to the cable to ensure there is no breakage, kinks, or tears. Inspect distal and proximal ends to ensure metal fittings are intact and undamaged.

Only compatible with a specific LED lightsource® manufactured by Sunoptic Technologies® and distributed by Thompson Surgical Instruments.

Use of non-LED and non-compatible light sources will void any warranty or service agreement and may increase the risk of patient injury.

Always use the Lite Wand LED and Retractor Lite LED with the Thompson Retractor as the Thompson Retractor allows the Lite Wand LED and Retractor Lite LED to be positioned in a way that prevents the distal end from being covered by skin or flammable materials which may cause burning or ignition.

The Retractor Lite LED and Lite Wand LED connect to a concentrated light source (luminous power per area) and this light energy is directed through, retained in and emitted from these instruments. The light output of these connected instruments if used in contact, close proximity or directed towards tissue or ignitable or flammable materials may present a risk of burn, injury or fire depending upon application. Qualified personnel must continually determine a safe working distance of the instrument during use and determine the proper intensity settings of the connected light source. The light output should never be left on unattended or misdirected according to this advisement.

FIRE HAZARD: Never drape or cover the end of the Lite Wand LED and Retractor Lite LED with anything, particularly materials which may ignite or are flammable.

Always set the intensity control to the minimum level before turning on the power and ensure the Lite Wand LED or Retractor Lite LED is positioned safely for the application.

When light is not required at the surgical site, the light source should be turned off or the intensity control should be set to the fully dimmed position to reduce energy output while continuing to implement safe working distances of the Lite Wand LED and Retractor Lite LED.

If it becomes necessary to move the Lite Wand LED or Retractor Lite LED without turning the light source off, turn the intensity control to the fully dimmed position.

User is responsible for determining if interruption of light output will create an unacceptable risk. If this determination is made, arrangements should be taken to reduce the risk.

Always turn the light source off before removing Lite Wand LED and Retractor Lite LED from the Thompson Retractor.

Lite Wand LED and Retractor Lite LED proximal end may be hot to touch after use. Use care when unplugging from the illuminator.

Never pry, tug, or yank the Lite Wand LED and Retractor Lite LED from the Thompson Retractor. Use care when separating the Lite Wand LED and Retractor Lite LED and the Thompson Retractor.

Do not bend Lite Wand LED and Retractor Lite LED cable to less than a $\frac{3}{4}$ " radius to prevent fiber breakage.

Do not use automatic cleaning machines as they may cause fiber breakage.

Do not point the light from the Lite Wand LED and Retractor Lite LED in a person's eye as it will cause discomfort. Ensure lightsource is turned off when handling if not in use on the Thompson Retractor.

Do not lever off the Lite Wand LED and Retractor Lite LED during use with instrumentation. Use care to avoid damaging the distal end when passing instruments through the Thompson Retractor. Damage may result in light transmission loss and fiber breakage.

ADVERSE EVENTS

We have no knowledge of any adverse events with this product.

CLEANING AND STERILIZATION FOR LITE WAND LED AND RETRACTOR LITE LED

The cleaning and sterilization methods for the devices were validated and information on them is provided under consideration of ISO 17664.

Lite Wand LED and Retractor Lite LED are high quality optical devices. They require similar care to that taken for any precision optical component.

During cleaning, care must be exercised handling the light guides to avoid breaking fiber. Do not forcefully bend, kink, or handle roughly.

CLEANING

Cleaning solutions must be prepared according to the manufacturer’s instructions and compatible with the material components (silicone, stainless steel, and glass) of the light guide.	
Step 1:	Rinse the light guides in running water to remove all loose or soluble contamination.
Step 2:	With fresh, non-protein fixing, cleaning solution, hand wash the Light guides.
Step 3:	Using a soft, non-metallic, brush and lint free towels remove all soil from the light guide. Special attention should be paid to the junctions between flexible and solid parts to remove any trapped soil.
Step 4:	Once clean, thoroughly rinse in running water to remove all traces of the cleaning solution
Step 5:	Rinse in distilled water to remove any dissolved minerals in the cleaning water.
Step 6:	Clean the fiber ends by gently rubbing with a cotton swab saturated in alcohol. Immediately dry using compressed air.

STERILIZATION - STEAM AUTOCLAVE

Sterilization can be accomplished in an AAMI ST8 compliant steam sterilizer with the following criteria:

Preparation: SPSmedical Self-Seal pouch SSP387 with SPSmedical STEAM*Plus* Integrator SSI-100

METHOD	CYCLE	MIN. TEMP.	EXPOSURE TIME	DRY TIME
Steam (in pouch)	Prevacuum	Prevacuum 270 deg. F (132 deg. C)	4 minutes	40 minutes

The above steam sterilization method was validated for these devices. If other equipment, methods or parameters are used, the user is responsible for validation.

LIMITED WARRANTY

Your Lite Wand LED or Retractor Lite LED has a one (1) year warranty from the date of shipment on workmanship and all defects of material except for broken fiber. Should your product prove to have such defects within one (1) year of shipment, Thompson Surgical Instruments, Inc. will repair or replace the product or component part without charge.

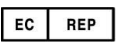
Should your Lite Wand LED or Retractor Lite LED need servicing under this warranty, please contact your distributor or your customer support specialist for return authorization documentation. You should carefully pack the product in a sturdy carton, including a note describing the defects, your name, your company name, telephone number and a return address. Warranty does not cover equipment subject to misuse, accidental damage, and normal wear and tear. This warranty gives you specific legal rights, and you may also have other rights that vary from state to state.

POST WARRANTY REPAIRS

Please contact your distributor or your customer support specialist for return authorization documentation.

Customer Service Telephone: 231-922-0177
www.thompsonsurgical.com

CHART OF MEDICAL DEVICE SYMBOLS USED

	Manufacturer
	Date of manufacture (YYYYMM or YYYY)
	“Authorized Representative in the European Community.”
	Caution, consult accompanying documents
	Caution: Federal law restricts this device to sale by or on the order of a (licensed healthcare practitioner).
	Batch code / lot number
	Consult Instructions for Use
	CE mark
	Nonsterile
	Catalog, reorder or reference number

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CE 0297

Distributor:

DIST Thompson Surgical Instruments, Inc.
10170 E. Cherry Bend Rd.
Traverse City, MI 49684

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