

Thompson Retractor

Adjustable Scope Grip (#33181)

Free Up Staff

Hold various sized scopes with the Adjustable Scope Grip when used in combination with Thompson laparoscopic holders. The Adjustable Scope Grip easily adjusts to fit 5-11mm scopes and holds scope firmly in position to free up staff.

BENEFITS

- Adjustable to hold various sized scopes
- Firmly holds and protects scope shaft
- Quickly connects to Thompson laparoscopic holders



"Thompson Surgical provides the most hassle-free, versatile retractor."

- Michael Nizzi, DO, FACOS




Step 1

Attach to scope shaft (5mm-11mm), rotating knob clockwise to tighten.



Step 2


Insert into laparoscopic holder distal end.

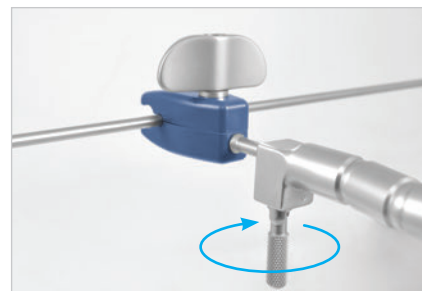

Step 3

Rotate hanging knob clockwise to secure and hold scope hands free.

 **ORDERING INFO:**
Adjustable Scope Grip 5mm - 11mm #33181


 **NOTICE:**
Recommended for use with the Articulating Arm or Flexible Scope Holder plus table mount.

 **Thompson**
SURGICAL INSTRUMENTS
10341 East Cherry Bend Road
Traverse City, Michigan 49684
phone: 231.922.0177
fax: 231.922.0174
thompsonsurgical.com

For a Free Trial Call **1-800-227-7543** *

* Free trial valid for U.S. customers only. Customers outside the U.S.A., please call +1-231-922-0177 for availability.

© 2017 Thompson Surgical Instruments, Inc. Traverse City, Michigan.

© S-Lock®, PLA®, and the Thompson "T Circle" logomark  are Registered Trademarks of Thompson Surgical Instruments, Inc.

Patents: US4971038, US5025780, US5888197, US5897087, US5902233, US5984865, US6033363, US6416465, US6511423, US7338442, US7749163, US8257255, US8360971, US8617064
Other patents pending.

EC REP

Emergo Europe
Prinsessegracht 20
2514 AP The Hague
THE NETHERLANDS

CE0297

Rev B
042817
asgss0417

**Made In
USA**