
## Femur Elevator

### Consistent Femur Elevation

The Thompson Retractor Femur Elevator uses a sterile mount Elite II Rail Clamp to securely attach to the OR table, providing stable elevation of the femur, eliminating the need for extra staff. The Femur hook and Micro-Adjustable handle allow for consistent and precise positioning.

*The Femur Elevator provides consistent, stable, and controlled exposure to the femur during hip procedure.*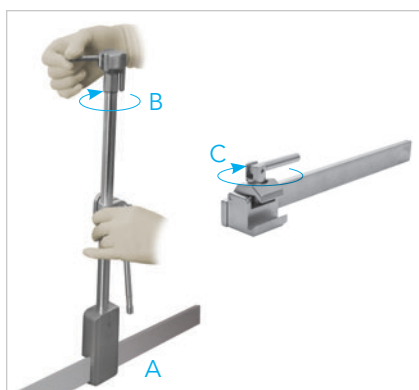


#### Step 1: Rail Clamp

Open rail clamp jaw and place over the sterile drapes (A).

Rotate the top knob clockwise to tighten (B), utilizing the attached hanging handles for leverage when needed.

*For Hana Table compatibility, add a 20" Rail Extender (#41938) to the table before attaching rail clamp.*


*To attach, open jaw and place over the sterile drapes. Rotate the knob clockwise to tighten (C).*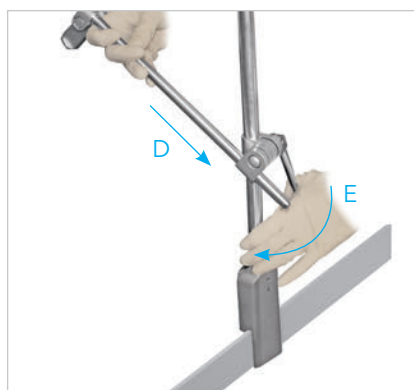

#### Step 2: Arm

Open the rail clamp cam joint and insert straight arm (D).

Position straight arm and flip the cam joint handle to lock in place (E).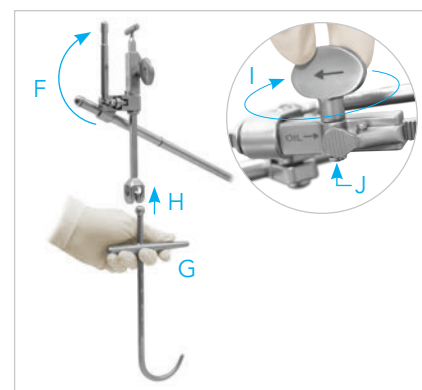


#### Step 3: Hook

Position Elevator Handle and lock in place by closing the joint (F).

Use the T-handle at proximal end of the femur hook (G) to position hook in place. Insert ball into elevator arm handle (H) to hold in place, hands-free.

If additional retraction is necessary, utilize the micro-adjustable handle by turning the knob to retract further (I).

Press pawl to release retraction (J).

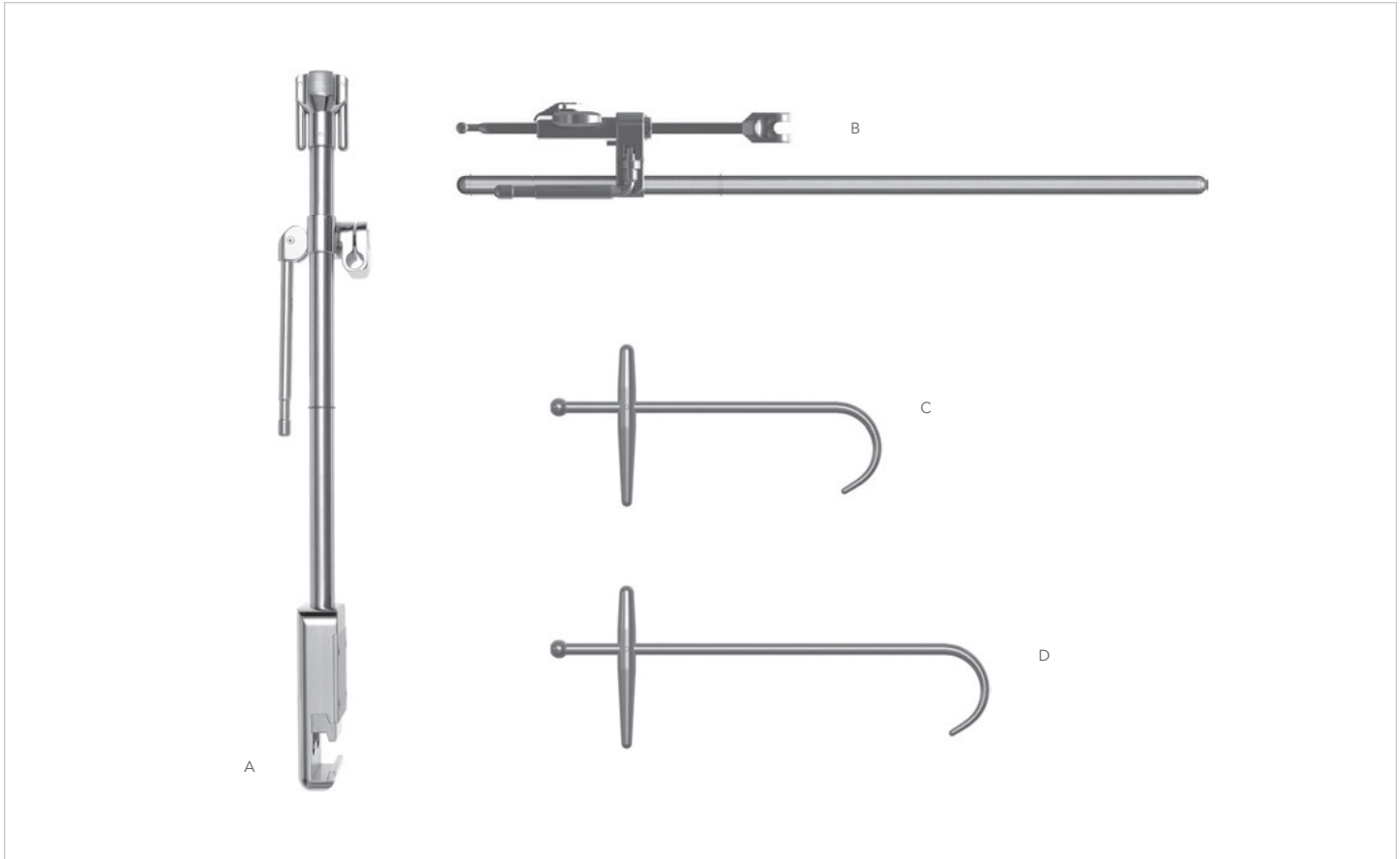**FEMUR ELEVATOR KIT**

**#90025**

REF	QTY	ITEM DESCRIPTION	PART #
A	1	Elite II Rail Clamp w/1 Cam Joint 18"	41902AC
B	1	20" Straight Arm w/Elevator Handle	44300
C	1	Femur Hook 9"	44309
D	1	Femur Hook 12"	44312
	1	Instrument Case w/Pin Mat 22" x 11" x 3 1/2"	50000G

**i KIT NOTICE**  
Femur Elevator Kit may be used as a stand-alone retractor. Hip Retractor Frame ( #SL90023 ) not required for use.

**i FRAME NOTICE**  
Add a Rail Extender ( #41938 ) for compatibility with Hana® tables or leg positioning systems such as the Arch system.

**Thompson**  
SURGICAL INSTRUMENTS  
10341 East Cherry Bend Road  
Traverse City, Michigan 49684  
phone: 231.922.0177  
fax: 231.922.0174  
[thompsonsurgical.com](http://thompsonsurgical.com)

EC REP **Emergo Europe**  
Prinsessegracht 20  
2514 AP The Hague  
THE NETHERLANDS

\* Free trial valid for U.S. customers only. International customers please call for more information.

© 2017 Thompson Surgical Instruments, Inc. Traverse City, Michigan.

® S-Lock®, PLA®, and the "T Circle" logomark are Registered Trademarks of Thompson Surgical Instruments, Inc.  
Patents: US4971038, US5025780, US5888197, US5897087, US5902233, US5984865, US6033363, US6416465, US6511423, US7338442, US7749163, US8257255, US8360971, US8617064  
Other patents pending.

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