

# Thompson Retractor

## Adjustable Scope Grip ( #33181 )

### Free Up Staff

Hold various sized scopes with the Adjustable Scope Grip when used in combination with Thompson laparoscopic holders. The Adjustable Scope Grip easily adjusts to fit 5-6mm and 10-11mm scopes and holds scope firmly in position to free up staff.


#### BENEFITS

- Adjustable to hold various sized scopes
- Firmly holds and protects scope shaft
- Quickly connects to Thompson laparoscopic holders



*"Thompson Surgical provides the most hassle-free, versatile retractor."*

**- Michael Nizzi, DO, FACOS**

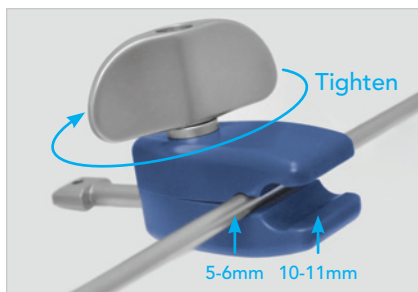

#### Step 1

Attach to scope shaft (5-6 or 10-11mm), rotating knob clockwise to tighten.

**NOTE: DO NOT OVERTIGHTEN**



#### Step 2


Insert into laparoscopic holder distal end.



#### Step 3

Rotate hanging knob clockwise to secure and hold scope hands free.

 **ORDERING INFO:**  
Adjustable Scope Grip 5-6mm / 10-11mm #33181


 **NOTICE:**  
Recommended for use with the Articulating Arm or Flexible Scope Holder plus table mount.

 **Thompson**  
SURGICAL INSTRUMENTS  
10341 East Cherry Bend Road  
Traverse City, Michigan 49684  
phone: 231.922.0177  
fax: 231.922.0174  
[thompsonsurgical.com](http://thompsonsurgical.com)

For a Free Trial Call **1-800-227-7543** \*

\* Free trial valid for U.S. customers only. Customers outside the U.S.A., please call +1-231-922-0177 for availability.

© 2020 Thompson Surgical Instruments, Inc.

© S-Lock® ®, PLA®, and the Thompson "T Circle" logomark  are Registered Trademarks of Thompson Surgical Instruments, Inc.

Patents: [trpat.com](http://trpat.com)

Other patents pending.

EC REP

**Emergo Europe**  
Prinsessegracht 20  
2514 AP The Hague  
THE NETHERLANDS

 0297

Rev E  
110620  
asgss

**Made In  
USA**