

Lite Wand Xe

Replace your headlight

The Lite Wand Xe easily attaches anywhere on your Thompson Retractor frame, while the bendable gooseneck allows light to reach the deepest cavities improving illumination where it is needed most. May also be attached to any Thompson Retractor blade with the optional Lite Clip.



Utilize the bright, cool, reusable Lite Wand Xe to add illumination to your procedures.




Step 1

- A** Clip the Lite Wand Xe joint anywhere on the retractor frame that will allow the distal end of the light to illuminate deep into the incision.
- B** Secure in place by locking the cam joint.



Step 2

Bend the gooseneck to position the distal end of the Lite Wand Xe above the incision.

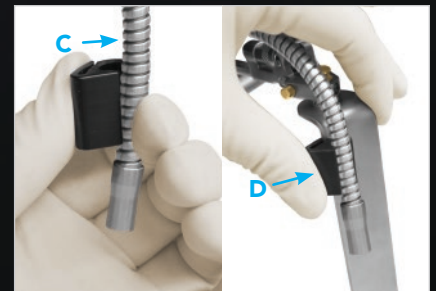

Step 3 (optional)


- C** Attach Lite Clip to gooseneck of Lite Wand Xe, with the thin opening toward the back of the lite.
- D** While applying pressure to the blade for leverage, slide the Lite and Lite Clip onto the desired blade. Push all the way onto blade to secure.



NOTICE:

Be sure that the bulb in your hospital's xenon light source is properly functioning, is turned up all of the way, and doesn't need to be replaced. A bulb that has been used over 700 hours can reduce light by up to 30%. Also, replacing bulbs with less expensive "off-brand" bulbs can cause light reductions in lighting accessories such as ours. Light sources should be Xenon only, 300W. Lastly, check to make sure your light source properly fits our Lite Wand Xe plugin. An improper fit will reduce light emissions.

We provide adapters for Olympus, Storz, and Wolf light sources if needed.

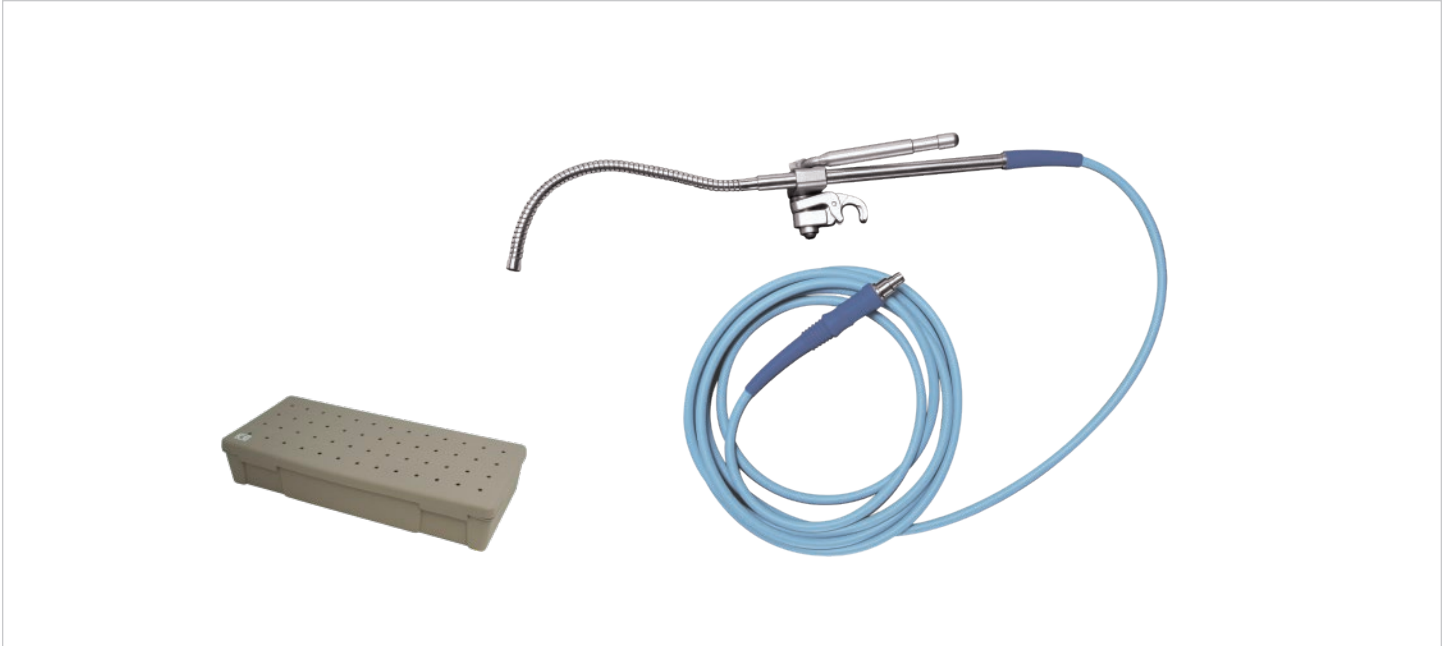


LITE WAND XE KITS *

ITEM DESCRIPTION	PART #
Lite Wand Xe with ACMI Cable	40001X
Lite Wand Xe with Storz Cable	40002X

ITEM DESCRIPTION	PART #
Lite Wand Xe with Wolf Cable	40004X
Lite Wand Xe with Olympus Cable	40019X

* Kits include Lite Wand Xe and Instrument Case.

 **NOTICE:**
Xenon light source (not included) required for use.

Thompson 
SURGICAL INSTRUMENTS
10341 East Cherry Bend Road
Traverse City, Michigan 49684
phone: 231.922.0177
fax: 231.922.0174
thompsonsurgical.com

* Free trial valid for U.S. customers only. International customers please call for more information.

© 2018 Thompson Surgical Instruments, Inc. Traverse City, Michigan.

© S-Lock® @, PLA®, and the "T Circle" logomark  are Registered Trademarks of Thompson Surgical Instruments, Inc.

Patents: US4971038, US5025780, US5888197, US5897087, US5902233, US5984865, US6033363, US6416465, US6511423, US7338442, US7749163, US8257255, US8360971, US8617064

Other patents pending.