


Thompson Femur Elevator User Manual:
SET UP INSTRUCTIONS FOR THOMPSON FEMUR ELEVATOR

Femur Elevator Set Up Instructions

Thompson Retractor 

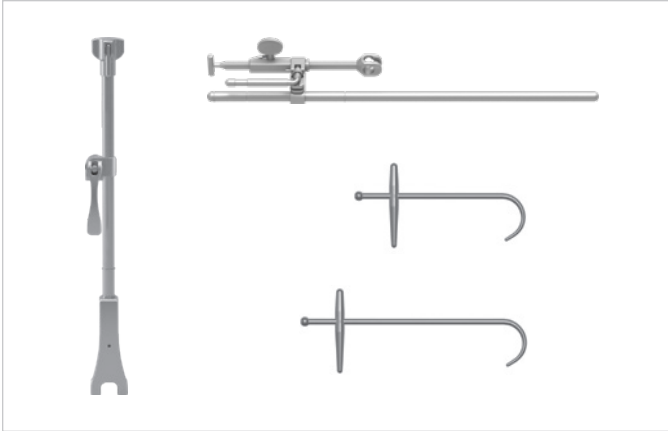

CAUTION: Avoid compressing the patient's body with frame components to prevent nerve damage.




The Thompson Retractor is provided in a non-sterile condition. Refer to Thompson Retractor IFU for cleaning, sterilization, and care instructions.

Reference the Thompson Retractor IFU for additional warnings and cautions.

For other Thompson Retractor Systems, reference corresponding User Manual for each.

Symbol Legend:

 Manufacturer	 Authorized EC Rep	 CE 0297 CE Mark	 Warnings / Precautions	 Non-Sterile
--	---	--	--	---

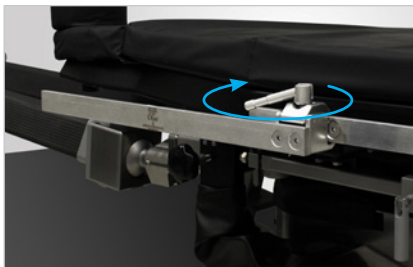


If Using a Hana® Table, or Leg positioning system, such as Arch

Place rail extender over the sterile drape and turn top knob clockwise to tighten.




Step 1: Rail Clamp

Place Elite Rail Clamp onto the rail over the sterile drape and turn top knob clockwise to tighten.

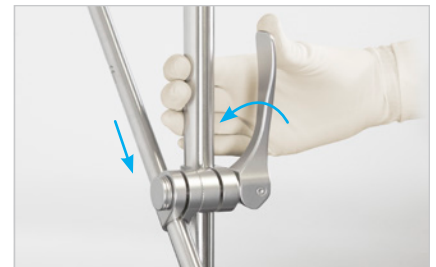


Step 2: Straight Arm

Insert the straight arm into the joint on the Elite Rail Clamp, position, and flip the cam joint handle to lock.

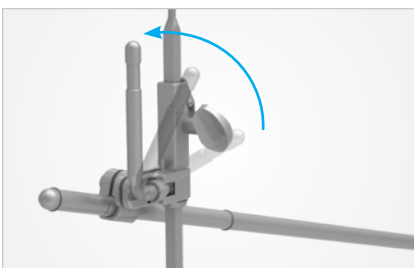


Step 3: Elevator Handle

Position Elevator Handle and lock in place by closing the joint.

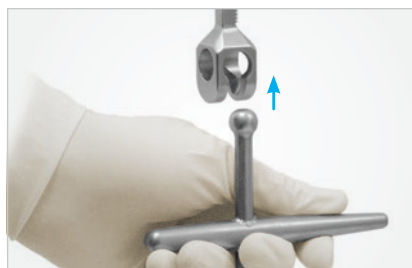

Step 4: Hook

Use the T-handle at proximal end of the femur hook to position hook in place. Insert ball into elevator arm handle to hold in place.

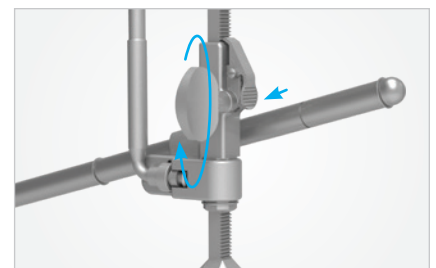

Step 5: Retract

If additional retraction is necessary, utilize the micro-adjustable handle by turning the knob to retract further. Press pawl to release retraction.

 **Thompson**
SURGICAL INSTRUMENTS
10341 East Cherry Bend Road
Traverse City, Michigan 49684
phone: 231.922.0177
fax: 231.922.0174
thompsonsurgical.com

EC REP

Emergo Europe
Prinsessegracht 20
2514 AP The Hague
THE NETHERLANDS

CE 0297